



**WESTMINSTER**  
INTERNATIONAL LTD

Telephone : +44 (0)1295 756300  
 Fax : +44 (0)1295 756302  
 E-Mail : info@wi-ltd.com  
 Website : www.wi-ltd.com


## WG 3-Ply Disposable Medical Face Masks



The WG 3-Ply Disposable Medical Face Masks provide protection from airborne infectious allergens, bacteria, dust and contaminants by covering the nose and mouth. It provides breathable comfort with low respiratory resistance.

One size fits all persons, the soft flexible nose piece permits individual facial adjustment.

### 3-Ply Non-Woven Layer Disposable Face Mask

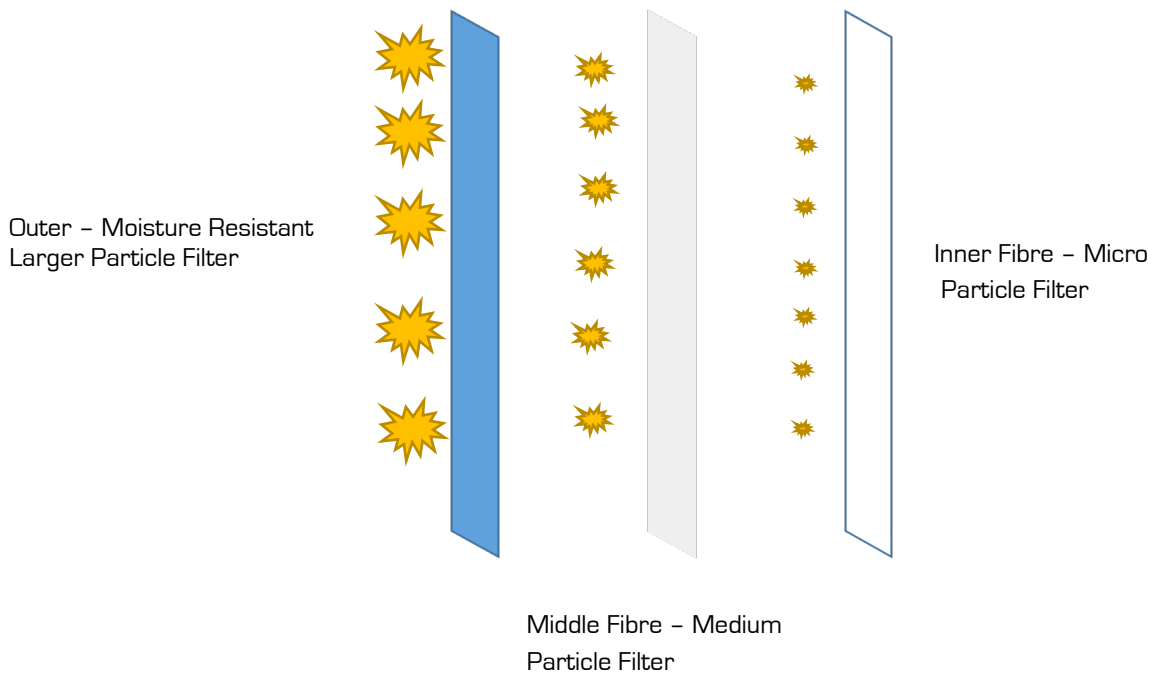

**Please note:** The wearing of a mask is an aid to help prevent the spread of airborne infectious allergens, bacteria, dust or contaminants it is not a guarantee and masks should always be used in accordance with official advice.

**Construction** - The outer layer is breathable non-woven and moisture resistant that will block larger dust and smoke particles, the middle layer is non-woven paper that will block medium size bacteria, dust and smoke particles, the non-woven micro fibre inner layer will block minute size bacteria, dust and smoke particles.

### Safe Practice Using the Mask

- Wash hands with warm water using antibacterial soap or antibacterial gel before handling the mask
- Identify the top of the mask, the metal strip is at the top
- Identify the and inner layer of the mask
- Apply the mask, passing the loops over the ears.
- Stretch the mask to cover the nose and mouth
- Squeeze the metal strip over the bridge of the nose, to provide a good close fit
- When removing the mask avoid contact with the outer layer.
- Change masks regularly throughout the day
- Do not reuse a mask
- Dispose of used masks safely